

THERMOMETER USE

- Please use a provided chux and alcohol-based sanitiser to clean thermometer when you arrive.
- If batteries have been removed or recently replaced, the instrument will require about 10 minutes before you can use it.
- The instrument is required to be held at a distance of 5 to 10 centimetres away from the forehead. Ask people to move their hair away if required and be aware if the person has heavy makeup that this may interfere with the result.
- Ask people to remain still until you tell them it is okay to move. It appears most of us anticipate an instant response and this is not necessarily the case.
- Hold down the “trigger” to take the temperature. Carefully hold the instrument in place until a temperature is recorded on the screen.
- Record the temperature on the questionnaire belonging to that member.
- A HIGH temperature reading (38°C or higher) will trigger a sound from the instrument. Please be alert if any temperature is close to the mark. Medications may reduce fever.
- A high temperature is a trigger to ask the member to please seek medical attention as soon as possible from their GP or the hospital – **IT IS OUR RESPONSIBILITY UNDER THE COVID-19 PROTOCOL TO DO SO. Entry to the Club WILL NOT be permitted.**
- **ALL** Questionnaires must be retained and stored for **EVERY** session in the appropriate Daily Register. This is a requirement for our Covid-19 Protocol
- Once all temperatures have been recorded for the session, please sanitise the instrument and store at the Entry table.
- Should the thermometer display show a Low battery warning, please replace with appropriate batteries stored under the Entry trolley. Note the time required until the instrument becomes operational – 10 minutes.